



Activator Pro

The Braille display for demanding workstations.
Efficient. Ergonomic. Without compromise.

The Braille revolution in the workplace

We have been developing Braille displays for over 30 years. With this experience, we asked ourselves: What would the perfect Braille display for the workplace have to look like to manage the ever-increasing flood of information as a blind person? Our answer: the Activator Pro. This ergonomically shaped Braille display with 64 or 80 concave Braille cells with adjustable dot firmness is not only impressively slim, but also offers pioneering innovations from unprecedented hardware and software solutions for increased workplace efficiency. See for yourself.



helptech.eu/braille-displays-activator-pro

**Keyboard with
Braille input**

**64 or 80
Braille cells**

**ActiveSplit &
BrailleShot**

**QuickAccess
keys**





More comfort at work

A particular highlight of the Activator Pro is the high-quality keyboard with integrated Braille input. The keys used are with top quality and precision. Thanks to modern mechanics, they ensure an incredibly pleasant typing experience.

The Activator Pro offers you an unprecedented level of customization. With the additional row of function keys (F13 - F24) and the 10 QuickAccess keys¹, the Activator Pro can be flexibly adapted to the requirements of your workplace at any time.

With a height of less than 2 cm, the Activator Pro is also impressively slim. Thanks to the ultra-compact and ergonomic design, even hours of work are always comfortable.

¹ Available with Activator Pro with 80 Braille cells.

Only from Help Tech: ActiveSplit

The ability for Braille displays to connect to multiple devices is nothing new. However, with the Activator Pro, it is now possible for the first time to control two devices simultaneously.

Thanks to our cutting-edge ActiveSplit technology, you can divide the Activator Pro Braille display into two segments - and assign each segment to its own device! For example, you want to display and control your PC and smartphone at the same time? This is easily possible with the Activator Pro.

The best part: With just two button presses, you can change the length of the two segments at any time.



Additional highlights

BrailleShot

BrailleShot is the equivalent of a screenshot for users of a Braille display. If you activate BrailleShot, the output on the Braille display is recorded as a text file. Combine BrailleShot with the DirectTransfer function to transfer the text created with BrailleShot to a connected device.

Active Tactile Control (ATC)

For the Activator Pro, we have further developed our patented ATC technology for automatic switching at the end of the line. ATC is now also used to determine the active Braille segment when ActiveSplit is activated. As a result, input elements such as the keyboard and QuickAccess keys are always assigned to the correct Braille segment, without you having to do anything!

Integrated shorthand translation

The translation and back-translation from full text to shorthand is directly integrated into the Activator Pro. We rely on RTFC, the world's leading translation tool for German, English and French. The Activator Pro also handles shorthand translation into Unified English Braille with ease.

The perfect addition: HelpTech+ for iPhone and iPad

The HelpTech+ app, available free of charge for iPhone and iPad in the App Store, continuously adds many new functions to the Activator Pro, making it a next-generation Braille experience.



Braille monitor

The integrated Braille monitor enables a sighted person to follow the information displayed on the Braille display directly on the iPhone or iPad display. Including reading position, text markings and the cursor position.



Braille dialogue

The Braille dialogue enables communication between deaf-blind and sighted people. The iPhone or iPad reads out the Braille input and the voice input on the iPhone or the on-screen keyboard can be used to respond.




Dictation function & speech output

With voice input, you can easily dictate texts instead of typing them yourself. The recognized text is then either saved in a new file on the Activator Pro or inserted into an existing file at the desired position. With the voice output function, you can have texts on the Activator Pro read aloud via the iPhone or iPad's speakers.



File manager & text transfer

With HelpTech+ you can copy files back and forth between the Activator Pro and your iPhone or iPad.

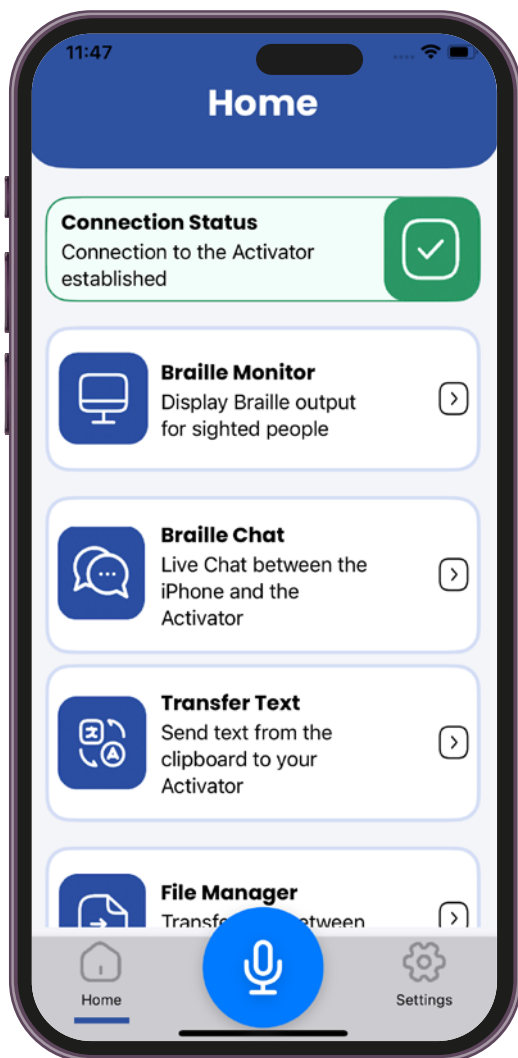

Download on the
App Store

It is our pleasure to consult you

Together with our international partner
we are there for you.



Help Tech GmbH & Co. KG
Brunnenstr. 10
72160 Horb - Germany



info@helptech.de
www.helptech.eu



+49 (0)7451 5546-0

Technical Data

- 64 or 80 concave Braille cells and adjustable dot firmness
- Keyboard with Braille input and 24 function keys
- 10 QuickAccess keys (Activator Pro 80 only)
- Activator Pro 80 dimensions (W x H x D): 56.6 x 1.9 x 23.7 cm
- Weight 80s version: 3,2 kg (7lbs)
- Activator Pro 64 dimensions (W x H x D): 51.1 x 1.9 x 23.7 cm
- Weight 64s version: 2,9 kg (6,4 lbs)
- Height of the keyboard: 0.4 cm
- Connections: 2 x USB-C, multi Bluetooth (up to 3 devices)

Activator Pro is a product of Help Tech.
Subject to changes © Help Tech 2024

